

FREEWAY

Assembly Instructions & User Guide

T90 & T90WC Paediatric

Shower / Commode / Toilet Chair

Dear Customer,

Thankyou, for purchasing a Freeway T90 / T90WC Paediatric shower chair from our extensive range of shower / commode / toilet chairs.

Please ensure that all the accessories you ordered have arrived with your chair, and all accessories are free from any damage.

Please read the following instructions carefully to enable you to use your chair in a safe manner, as they contain important safety information.

Note: The backrest forms part of the mainframe of the chair, so no assembly is required.


1. Fitting the Seat to the frame.

- Remove the plastic bag containing the two black hand nuts secured at the rear of the frame. Keep these as they are required for fitting the armrests and the footrests.
- Remove the plastic bag from the seat and place the seat onto the frame.
- Rotate the 4 plastic retention clips over the frame. Make sure they are rotated 90 degrees, to ensure the seat is secured correctly. (see photo)




2. Fitting the Armrests.

- Remove the armrests from their packaging and the 2 black hand nuts from the plastic bag.
- Insert armrest legs into the sockets, ensuring the rivet is at the front.
- Tighten the hand nuts to secure the armrests.

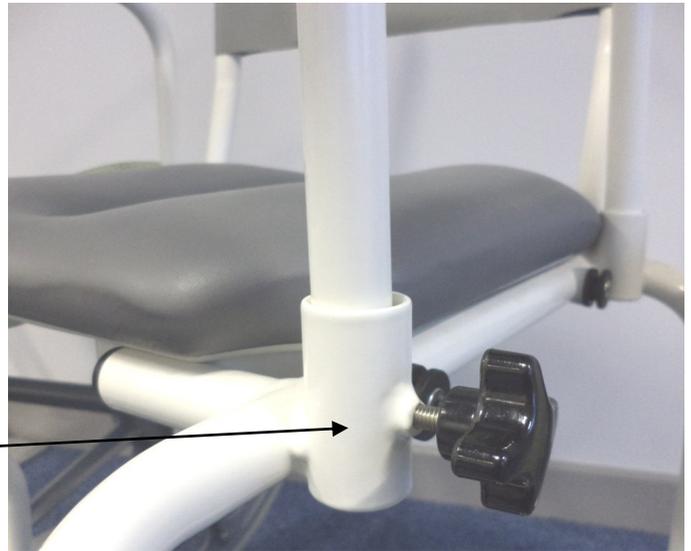

3. Fitting the Footrests.

- To fit the footrest onto the chair, slide the square tube into the frame tube.
- Use the black hand nut to secure the footrest to the frame. (see photo)

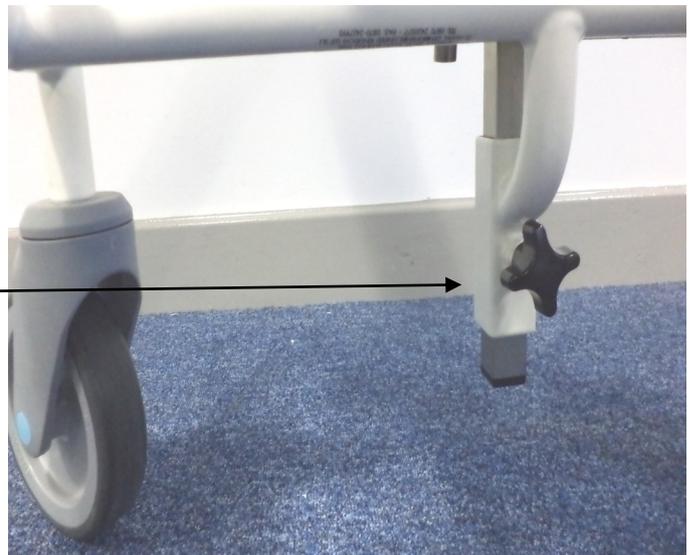

4 . Using the Brakes.

For T90 chairs fitted with large wheels the brakes are fitted at the front of the chair.

- The brakes are operated by pushing the brake lever down. To ensure that the brake is holding the chair correctly, ensure the lever is pushed to its stop. (see photo's)



SAFETY WARNINGS

- Ensure that all maintenance and checks are followed, (see page 4)
- Non self-propelled Chairs should be used when a carer is present.
- NEVER use your Freeway shower/ toilet chair without ensuring that the 4 retaining clips that secure the seat are in the “locked” position.
- Armrests are detachable to aid in transfer and storage only. Both Armrests should be fitted to the chair when in use.
- On self propelled models, when engaging the brake on the large wheels, ensure that the lever is fully depressed to its “stop.”
- When using the chair over a toilet, always ensure that the toilet lid & seat are up.

STABILITY

Your Freeway shower / commode chair has been manufactured to be safe, durable, reliable for use by persons up to the safe working load of that chair. All the T90 range of chairs are tested to 160 kgs (25 stone) and come with a standard 12 month warranty.

Please do not overload the chair or use it in any environment that could cause damage as this could adversely effect your guarantee, should any problems arise.

- Do not stand on the footrests, or allow the footrests to be stood upon.
- Do not sit upon or allow others to sit upon the armrests of the chair.
- During transfer, engage all brakes.

CLEANING

Your chair has been coated with Freecoat our unique anti-bacterial and anti-microbial coating. The coating is durable and prolongs the life of the chair.

- Wipe clean and dry as much as possible after use.
- Once a week or more often as necessary, clean with an all-purpose cleaning fluid or mild disinfectant, rinse, then dry as much as possible.

MAINTENANCE

BEFORE EACH USE:

- Check backrest nuts and seat securing clips are in position, footrests are securely on spigots, armrest locking nuts are in position and brake levers are secure.

EACH WEEK:

- As daily check
- Check that the castors turn without obstruction. Apply brakes and test function.
- Check wheels on self propelling models are free running without excessive play, and that the nut on the axle bolt is tight.
- Check seats for damage to vinyl or base that might allow ingress of water.
- Check backrest vinyl is secure and free from damage.

EVERY YEAR:

- The chair should be serviced every 12 months to check for wear, and to make any adjustments necessary.

DO NOT JET WASH CASTORS OR SEAT

DO NOT DROP SEAT

There is a large selection of optional extras that can be fitted to FREEWAY chairs.

See your agent for details.

Unit 1, Tir Llwyd Industrial Estate, St Asaph Avenue, Kinmel Bay, Rhyl, Denbighshire, LL18 5JA

Tel +44 (0) 844 980 2296

www.prismmedical.co.uk

info@prismmedical.co.uk