

freeway

# Shower Chairs



prismmedicaluk

a prismhealthcare company

# Quality. Durability. Hygienic

## Backrests and handles

All models are available with either horizontal push handles or vertical handles for use in restricted areas or over WCs. The backrest is pliable and moulds to the user while providing excellent support.

## Frames

Available in three sizes:

'A' - 540mm (21") wide

'B' - 490mm (19") wide

'C' - 440mm (17") wide

## High-performance wheels

With stainless steel bearings and puncture-proof tyres.

## Coating

All Freeway chairs are treated with an antibacterial and anti-microbial coating. The coating is durable and prolongs the life of the chair.



## Armrests

With soft but supportive tops. The armrests detach or swing away for easy access and transfer.

## Seats

Fully padded, horseshoe or full ring seats with different-sized apertures that are interchangeable.

## Brakes

Self-propelled models have large wheel brakes fitted as standard.

## Castors

Highest quality large diameter plastic castors with stainless steel bearings. Some chairs feature service-free, anti-tangle castors as standard.

## Footrests

They flip up and are height adjustable. They can also be easily removed or swung conveniently away from the user's foot area.

# Bespoke, cost-effective toileting and showering solutions

Manufactured in the UK from only the highest quality materials, the comprehensive range of Freeway assisted or self-propelled shower, toilet and commode chairs are designed to provide the most effective solution to a client's toileting and showering needs.

**Freeway Shower Chairs are modular** and can be configured to choose interchangeable seats, backrests, armrests and footrests. A variety of accessories are available to ensure the most bespoke solution.

All adult models within the range are specifically designed to fit over a standard toilet or commode, with three widths available; 'A' – 540mm (21"),

'B' – 490mm (19") and 'C' – 440mm (17"). Paediatric, plus-size and models to fit over automatic toilets are also included in the range.

The modular design of Freeway Shower Chairs makes the range one of the most cost-effective on the market, as accessories and parts can be added to suit a client's changing needs. Frames can be lowered/raised, rear wheels set back/forward, and the frame depth extended/shortened.

Where there are special needs, we can manufacture a bespoke shower chair subject to an individual assessment. All models have a 5-year warranty as standard.



# Shower Chairs



## Freeway T20 Moulded Shower Seat Chair

Available as a static model or with four castors. It has a moulded polypropylene seat and fixed arms.



## Freeway T30 Free Standing Shower Chair

The Freeway T30 is a static shower chair with slip-resistant rubber feet or floor-fixing options available.



## Freeway T40 Assistant-Propelled Shower Chair

Versatility is the key with the T40 as it can be raised/lowered, extended forward, and rear castors set back for amputee use, to name a few of its applications. Features four total stop castors and footrests as standard.

The **Freeway T40 COM** is a version of the standard T40 modified to work seamlessly with the Clos-o-Mat Samoa toilet.



The **Freeway T40 AUTO** is for use with automatic toilets, including the Mera, Geberit range and Clos-o-Mat Palma Vita. Features pram handle backrest and seat with a skirt as standard.

A version to work with the BIO bidet is also available.

All models except T20 are available as shower, toilet or commode chairs.



### Freeway T50 Front Wheel Self-Propelled Shower Chair

The Freeway T50 features 510mm (20") braked front wheels and includes footrests as standard. It is ideal for operating in tight spaces due to the positioning of the castors.



### Freeway T60 Self-Propelled Shower Chair

The short overall length makes this chair ideal for small shower cubicles. The chair's low braked wheel facilitates seamless side transfers as the top of the wheels is below the chair's seat height. Available as self-propelled or assistant-propelled, the 510mm (20") rear wheel aids manoeuvring over a lip or small step. Anti-tip bars are featured as standard.



### Freeway T70 Self-Propelled Shower Chair

The benefits of large braked rear wheels make this chair easy to self-propel. The standard self-propelled model with 560mm (22") rear wheels has easily removable push rims and footrests fitted.

**Freeway T70 COM:** Modified to work seamlessly with the Clos-o-Mat Samoa toilet.

**Freeway T70 AUTO:** Designed for use with automatic toilets, including the Geberit 8000 range and Clos-o-Mat Palma Vita

**Freeway T70 Side Access:** Equipped with left /right side opening and adapted swing-away armrests. A version to work with the BIO bidet is also available.

# Height Adjustable

## Freeway Height Adjustable Shower Chair

A highly versatile, height-adjustable assistant-propelled chair. The seat height can be adjusted from 510 – 560mm in 25mm increments. The chair features service-free, anti-tangle castors with red brake levers as standard. It benefits from the same bespoke configurations as the T40.



# Reclining

The T80 is available as a shower, toilet or commode chair.

## Freeway T80 Reclining Shower Chair

The Freeway T80 is highly versatile and reclines up to 40 degrees through infinite positions. The tilt action is easy and light to operate.

The Freeway T80 is available as assistant-propelled or self-propelled and has a safe working load of 160kg (25 stone).

An extensive range of accessories is available for this model, such as a full or half-height detachable headrest.



## Freeway T80 COM

A variation of the T80 reclining shower chair, designed to work seamlessly with the Clos-o-Mat Samoa toilet.



## Freeway T80 AUTO

A version of the T80 for use with automatic toilets, including the Mera, Geberit range and Clos-o-Mat Palma Vita.

## Freeway Flow Shower Cradle

The Freeway Flow offers the next level of support, providing a comfortable, controlled and secure bathing experience, which is particularly advantageous for those with reduced mobility.

The cradle features anti-tangle castors, removable fabric and a height-adjustable frame with three height settings across 25mm increments. It is available as a shower or commode chair.



# Paediatric

The Freeway Paediatric Shower Chair range features a seat width of 360mm and a seat depth of 325mm.



## Freeway T80 Paediatric Reclining Shower Chair

The Freeway T80 Paediatric is a reclining shower or toilet/commode chair that has been purpose-built for the needs of younger clients.



## Freeway T90 Paediatric Shower Chair

The Freeway T90 Shower Chair/Commode features a seat height of 460mm. Footplates are flip-up, removable and height/depth adjustable. The T90 is available as an assisted or self-propelled model.

The **Freeway T90 WC** is a variation of the above, but for usage as a toilet commode – with a seat height of 560mm, allowing clearance over standard toilets and automatic bidets. NB: Self-propelled model is wider due to the splayed frame

# Plus-Size



## Freeway T100 Bariatric Shower Chair

The Freeway T100 carries up to 350kgs (55 stone) and is suitable for various applications for bariatric clients. Seat dimensions are 790mm wide and 550mm deep.

The T100 benefits from increased diameter tubing, special heavy-duty castors, and heavy-duty footrests with heel supports which are height adjustable, removable and swing away. The T100 is also available in smaller width frames than the standard.

**All models except T20 are available as shower, toilet or commode chairs.**



# Adaptations & Accessories

Freeway Shower Chairs can have adaptations, accessories and alterations to suit specific needs. Some options shown here may be fitted to an existing chair, while others will be available only on a new build.

**We understand that there are many options available. Please take advantage of a free assessment so that we can find the right solution for you.**

## Armrests



Raised/  
lowered



Flip up (new  
orders only)



Gullwing (new  
orders only)



Extended  
forward



Gutter (with or  
without strap  
supports)



Padded style

## Seats



Horseshoe



Fully padded



Full ring

## Backrests



5° reclined



Extended



Detachable  
headrest

A padded seat is a major contributor to the comfort and support offered by any Freeway Shower Chair.

The vast choice of standard seats means there is a seat for most situations, and we also supply custom seats for exceptional cases.

The following are examples of custom seats:

- Increased or extra comfort padding
- Reduced padding
- Wedged padding
- Dartex upholstery (breathable four-way stretch fabric allows more significant moulding of seat to the user and may help with pressure relief)
- Different sized/shaped apertures
- Extended seat depths (over 75mm involves frame alteration)

## Footrests



Raised (60, 90 or 120mm higher)



Heavy duty



Integral heel supports with Velcro toe/foot straps



Clip-on padded foot tray




Individual clip-on foot tray



Wrap-around vinyl footrest covers



Dartex footplate sleeve

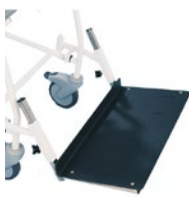

ABS foot tray



Padded ABS foot tray



Sandals foot holder (new footrest orders only)



Adjustable leg support and footrest



Adjustable angle footrest



Adjustable stump support



Double-padded stump support



Padded double-leg support (optional dimensions)

## Hygiene



Commode pan



Cup-shaped splashguard



Lower commode pan tray



Pommel (photo shows adapted for use on horseshoe seat)

# Straps, Harnesses and Positional Aids



VELCRO® Brand fastener waist strap



Waistband buckle fastening



Full harness



Ankle strap



Vinyl calf support



Armrest padding



Clip-on padding for backrests and armrests



Adjustable lateral supports (extended backrests only)



Padded trunk supports



Unpadded trunk supports



Knee block (new chairs only)



Black curved thoracic support (extended backrests only)



Curved thoracic wedges (retrofit kit available for all T80 backrests)



Curved padded head support

# Miscellaneous



Counterbalance weights for double amputee



Extended brake levers



Capstans on wheel












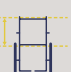




T30 floor fixing kit

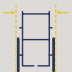
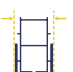





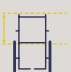






# Technical Specifications






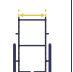



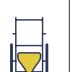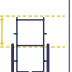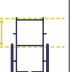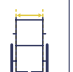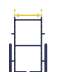

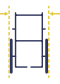


Below are the technical specifications for the most popular Freeway Shower Chair models. Please consult our website, [www.prismmedical.co.uk](http://www.prismmedical.co.uk), for the full range of dimensions.













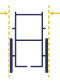


## Standard Chairs








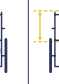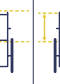




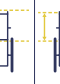


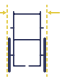These chairs have been weight tested to 200kg (30 stone), except for the T100, which is weight tested to 350kg (55 stone).


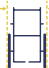








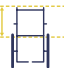
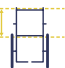
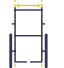

	<b>1</b> - Overall maximum height		<b>8</b> - Standard depth of the seat
	<b>2</b> - Overall maximum width		<b>9</b> - Top of the armrest to top of the seat
	<b>3</b> - Length with footrests fitted at the lowest position		<b>10</b> - Width clearance
	<b>4</b> - Length with footrests fitted at the highest position		<b>11</b> - Top of backrest handle to top of the seat
	<b>5</b> - Width of standard seat		<b>11a</b> - Top of backrest vinyl to top of the seat
	<b>6</b> - Height from floor to top of the seat		<b>12</b> - Distance between armrest at the rear
	<b>7</b> - Height clearance		<b>13</b> - Distance between armrest at the front



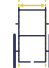



T30														
	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>11a</b>	<b>12</b>	<b>13</b>
<b>T30A</b>	1025	660	570	570	540	480	355	400	190	485	490	450	475	520
<b>T30B</b>	975	610	570	570	490	480	355	400	190	435	490	450	425	470
<b>T30C</b>	925	560	570	570	440	480	355	400	190	385	490	450	375	420








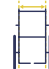

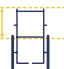


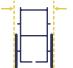


<b>T40</b> 														
	1	2	3	4	5	6	7	8	9	10	11	11a	12	13
<b>T40A</b>	980	640	1050	950	540	560	445	400	190	485	490	450	475	520
<b>T40B</b>	980	610	1050	950	490	560	445	400	190	435	490	450	425	470
<b>T40C</b>	980	560	1050	950	440	560	445	400	190	385	490	450	375	420
<b>T40A AUTO</b>	980	640	1050	950	540	560	480	400	190	485	490	450	475	520
<b>T40B AUTO</b>	980	610	1050	950	490	560	480	400	190	435	490	450	430	470
<b>T40C AUTO</b>	980	560	1050	950	440	560	480	400	190	385	490	450	410	410
<b>T40A COM</b>	980	640	1050	950	540	560	470	400	190	485	490	450	475	520
<b>T40B COM</b>	980	610	1050	950	490	560	470	400	190	435	490	450	425	470
<b>T40C COM</b>	980	560	1050	950	440	560	470	400	190	385	490	450	375	420









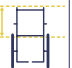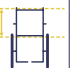


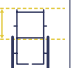
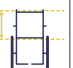

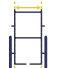
<b>T50</b> 														
	1	2	3	4	5	6	7	8	9	10	11	11a	12	13
<b>T50A</b>	980	730	960	930	540	560	445	400	190	485	490	450	475	520
<b>T50B</b>	980	680	960	930	490	560	445	400	190	435	490	450	425	470
<b>T50C</b>	980	630	960	930	440	560	445	400	190	385	490	450	375	420

<b>T60</b> 														
	1	2	3	4	5	6	7	8	9	10	11	11a	12	13
<b>T60A</b>	980	690	1060	960	540	560	445	400	190	485	490	450	475	520
<b>T60B</b>	980	640	1060	960	490	560	445	400	190	435	490	450	425	470
<b>T60C</b>	980	590	1060	960	440	560	445	400	190	385	490	450	375	420

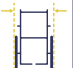T70														
	1	2	3	4	5	6	7	8	9	10	11	11a	12	13
T70A	980	750	1100	1150	540	560	445	400	190	485	490	450	475	520
T70B	980	700	1100	1150	490	560	445	400	190	435	490	450	425	470
T70C	980	650	1100	1150	440	560	445	400	190	385	490	450	375	420
T70A AUTO	980	790	1060	1140	540	560	480	400	190	485	490	450	475	520
T70B AUTO	980	740	1060	1140	490	560	480	400	190	435	490	450	430	470
T70C AUTO	980	690	1060	1140	440	560	480	400	190	385	490	450	410	410
T70A COM	980	775	1020	1110	540	560	470	400	190	485	490	450	475	520
T70B COM	980	725	1020	1110	490	560	470	400	190	435	490	450	425	470
T70C COM	980	675	1020	1110	440	560	470	400	190	385	490	450	375	420

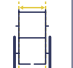T90 Paediatric														
	1	2	3	4	5	6	7	8	9	10	11	11a	12	13
T90 Assistant-propelled	910	620	690	N/A	360	440	N/A	330	140	N/A	480	440	300	340
T90 Self-propelled	910	620	740	N/A	360	440	N/A	330	140	N/A	480	440	300	340
T90WC Assistant-propelled	980	530	740	N/A	360	560	440	330	160	440	480	440	320	325
T90WC Self-propelled	980	690	870	N/A	360	560	440	330	160	440	480	440	320	325
















<b>T100 Plus-Size</b> 														
	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>11a</b>	<b>12</b>	<b>13</b>
<b>T100</b>	970	860	1220	1170	790	560	440	550	190	740	480	440	720	750
<b>T100FR</b>	970	910	1220	1170	790	560	440	550	190	740	480	440	720	750












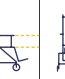




<b>Height Adjustable</b> 														
	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>11a</b>	<b>12</b>	<b>13</b>
<b>Height Adjustable A</b>	980	640	1050	970	540	560	445	400	190	485	490	450	475	520
<b>Height Adjustable B</b>	980	610	1050	970	490	560	445	400	190	435	490	450	425	470
<b>Height Adjustable C</b>	980	560	1050	970	440	560	445	400	190	385	490	450	375	420

*This frame adjusts in height. The floor to top of seat measurement can be 510, 535 or 560mm.*

# T80 Chairs

The T80 has been weight tested to 160kg (25 stone).

	<b>1 - Height</b>		<b>7 - Height clearance</b>
	<b>2 - Height with detachable headrest</b>		<b>8 - Seat depth (standard seat)</b>
	<b>3 - Width</b>		<b>9 - Top of the armrest to top of the seat</b>
	<b>4A - Length in an upright position</b>		<b>10 - Width clearance</b>
	<b>4B - Length with max. recline, including footrests at the highest position</b>		<b>11 - Top of the backrest to top of the seat (without detachable headrest fitted – with headrest add 300mm)</b>
	<b>4C - Length max. recline and detachable headrest fitted</b>		<b>12 - Distance between armrests at rear</b>
	<b>5 - Seat width</b>		<b>13 - Distance between armrests</b>
	<b>6 - Height to the top of the seat</b>		

T80 															
	1	2	3	4A	4B	4C	5	6	7	8	9	10	11	12	13
<b>T80A</b>	1140	1460	650	900	1150	1300	540	570	445	400	190	480	535	475	520
<b>T80B</b>	1140	1460	600	900	1150	1300	490	570	445	400	190	430	535	425	470
<b>T80C</b>	1140	1460	550	900	1150	1300	440	570	445	400	190	380	535	375	420
<b>T80A COM</b>	1140	1460	730	900	1150	1300	540	570	470	400	190	480	535	475	520
<b>T80B COM</b>	1140	1460	680	900	1150	1300	490	570	470	400	190	430	535	430	470
<b>T80C COM</b>	1140	1460	630	900	1150	1300	440	570	470	400	190	380	535	410	410
<b>T80A AUTO</b>	1140	1460	730	900	1150	1300	540	570	470	400	190	480	535	475	520
<b>T80B AUTO</b>	1140	1460	680	900	1150	1300	490	570	470	400	190	430	535	425	470
<b>T80C AUTO</b>	1140	1460	630	900	1150	1300	440	570	470	400	190	380	535	375	420
<b>T80P</b>	1024	1320	560	760	1060	1220	360	570	470	320	160	500	510	400	400



# Freeway Flow

The cradle has been weight tested to 130kg (20 stone).

**A** - Seat height

**F** - Back length

**J** - Internal width

**B** - Cradle end height

**G** - Seat length

**K** - Overall width

**C** - Handle height

**H** - Leg length

**L** - Upright length

**D** - Overall height

**I** - End length

**M** - Reclined length

**E** - Base frame length

Freeway Flow	A*	B	C*	D*	E	F	G	H	I	J	K	L	M
Small	592	176	940	1106	800	620	300	360	150	440	502	950	1260
Medium	592	116	1050	1214	800	750	350	450	150	440	502	1134	1508
Large	592	116	1050	1214	800	750	350	450	150	520	582	1134	1508

All measurements in mm.

\*Measurements taken at lowest height setting.



Thank you for taking the time to read our brochure. We hope you found it informative and helpful in selecting the right shower chair for your needs. Did you know that we also produce a variety of

shower stretchers and trolleys built to the same high standards of quality and durability? These products provide safe and comfortable support for individuals with limited mobility during bathing and showering.

## A sample of our Shower Stretchers and Trolleys



Please visit our website or speak to your local representative to learn more about our products. Our commitment to providing innovative

solutions that improve quality of life and promote independence for individuals with mobility challenges is at the heart of everything we do.

To request a quote or free assessment, please call us on **01924 840 100** or visit [www.prismmedical.co.uk/contact-us](http://www.prismmedical.co.uk/contact-us)

Ref:01/24

**Prism Medical UK Head Office**

Unit 4, Jubilee Business Park,  
Grange Moor, Wakefield  
WF4 4TD

**Prism Medical UK Manufacturing Centre**

Unit 1, Tir Llwyd Industrial Estate,  
Kinmel Bay, Conwy  
LL18 5JZ

Telephone: **01924 840100**

Email: [info@prismmedical.co.uk](mailto:info@prismmedical.co.uk)

Website: [www.prismmedical.co.uk](http://www.prismmedical.co.uk)

Download our shower chair configurator



**prismmedicaluk**  
a prismhealthcare company